
J10 demo board 使用手册

www.jhearing.com

2023. Nov.

1. J10 demo board 示意图

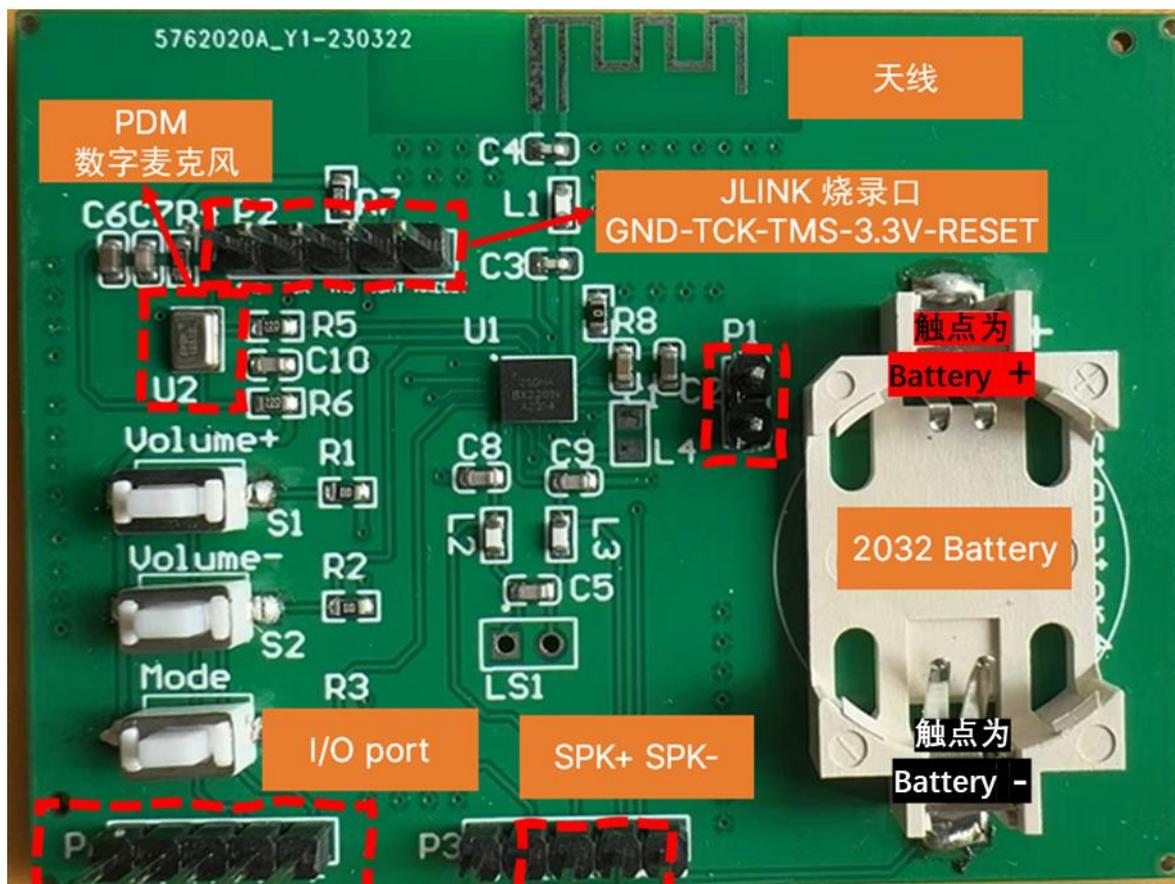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J10 Hearing Aid ,you can fitting on the web page

Support On following OS:

- Windows 10+ ,Edge Browser /Chrome Browser
- Linux ,Mac ,see [implementation status](#)
- Android Chrome
- Iphone/iPad : Please install "Bluefy – Web BLE Browser" first

if you have issues about web ble,please check implementation status

All BLE Devices Scan J10 Hearing Aid

Current Mode: Mode A Read Mode Config Save Mode Config Restore Mode Config Reset Mode Config Backup All Modes Test Change Volume

General EQ WDRC Calibration and Others Ble Upgrade

yp.jhearing.com/webj10_2022.php

WECHAT FITTING GITHUB CONTACT US

on the web page

All BLE Devices Scan J10 Hearing Aid

Restore Mode Config Reset Mode Config Backup All Modes Test Change Volume

#	Volume	[0-60]	This parameter only applicable for version after 2023-06-25
#	Power on delay	0 Seconds	After x seconds ,the DMIC will turn on to work
#	Enabled Modules	None	
#	Hardware Revision	J10	J10/J11 . Our firmware consider J10 as default hardware , You need reboot your hearing aid to make effect. for Custom PCBA,we will deliver customized firmware for you
#	BLE Display Name	0	Only letter and digits allowed , maximum length is 10 bytes ,You need reboot your hearing aid to make effect

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Current Mode:

General **EQ** WDRC Calibration and Others

Name	Value	
# ADC Gain	<input type="text" value="0 dB"/>	
# DAC Gain	<input type="text" value="0 dB"/>	
# Power on delay	<input type="text" value="0"/> Seconds	After x seconds ,the DMIC will turn on to work
# Enabled Modules	<input type="text" value="None"/>	Attention: after you enable WDRC ,please decrease DACGain or Output Driver Gain ,Otherwise your may get unexpected too loud voice
# Hardware Revision	<input type="text" value="J10"/>	J10/J11/J112023, , Our firmware consider J10 as default hardware , You need reboot your hearing aid to make effect
# BLE Display Name	<input type="text" value="0"/>	Only letter and digits allowed , maximum length is 9 bytes , You need reboot your hearing aid to make effect

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